
Human Control serum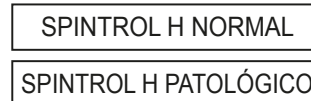

Suero Control humano

**Serum control multicomponent for clinical chemistry assays  
IVD**

Store at 2 - 8°C.

**PRODUCT CHARACTERISTICS**

SPINROL H NORMAL or PATHOLOGIC is a human lyophilised serum. With most constituent concentrations and activities in the normal or pathologic range. It's intended for control of accuracy for use with manual and automated analytical procedures.

**REAGENTS**

Human serum. Biological additives. Bacteriostatics agents. The concentration/ activities of the components are lot-specific. The exact values and ranges valid for reagents are given informational purpose in the value sheet.

**PRECAUTIONS**

The individual units comprising lot or lots of components from human origin, have been tested for HBsAg and antibodies to HIV and HCV found negative / non reactive by an FDA approved test. However, handle cautiously as potentially infectious.

**PREPARATION**

-Reconstitute (→) with 5,0 mL of distilled water.  
-Mix thoroughly, avoiding foam forming.  
-Bring to room temperature for about 30 min. before use.  
Improper handling and/or storage can affect results. Inaccurate reconstitution and errors in assay technique can cause erroneous results.

**STORAGE AND STABILITY**

The Control serum is stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use. Do not use reagents over the expiration date or if there is visible evidence of microbial growth.

Store tightly capped when not use.

-After reconstitution is stable for:

At 15°C to 25°C 12 hours. At 2°C to 8°C 5 days. At -25°C to -15°C 28 days (when frozen once).  
-Bilirubin and Acid Phosphatase (stored protected from light):  
At 15°C to 25°C 8 hours. At 2°C to 8°C 24 hours. At -25°C to -15°C 2 weeks (when frozen once).  
-ALT:  
At 15°C to 25°C 12 hours. At 2°C to 8°C 5 days. At -25°C to -15°C 2 weeks (when frozen once).

**PACKAGING**

SPINROL H NORMAL  
SPINROL H PATHOLOGIC

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SPINREACT, S.A./S.A.U. Ctra.Santa Coloma, 7 E-17176 SANT ESTEVE DE BAS (GI) SPAIN